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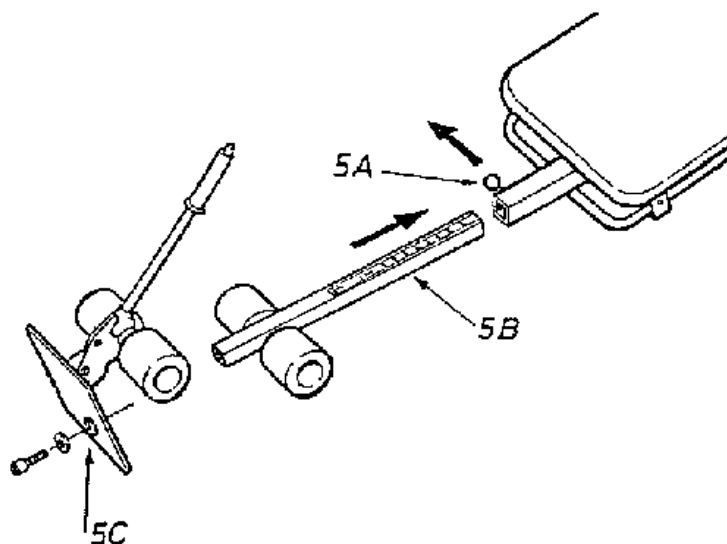
Master-care®
The Swedish Back Care System™

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(CN-A1, CN-A2, CN-A3, CN-AH6 and/y CN-B1, CN-B2, CN-B3)

Manufactured under license for Swedish Backcare System, Inc.
Bajo licencia de manufactura para Swedish Backcare System, Inc.

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ASSEMBLY INSTRUCTIONS *MASTERCARE - BACK-A-TRACTION*, ALL CN-A, CN-B MODELS

IMPORTANT! ALWAYS FOLLOW THE INSTRUCTIONS

WARNING! KEEP HANDS AND FINGERS CLEAR OF WHEELS ON SLIDING BACKREST!

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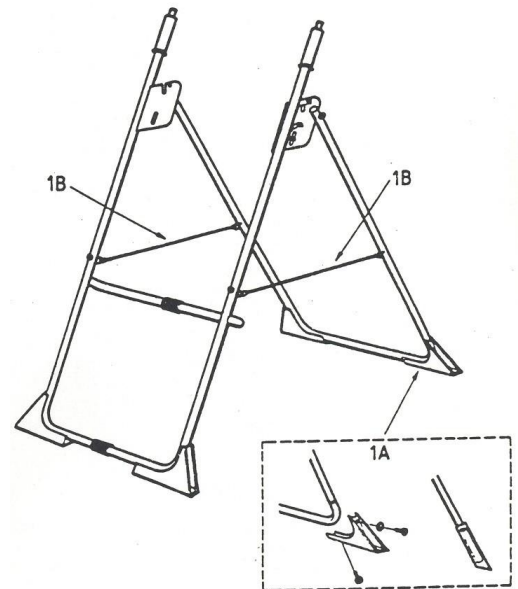
MOUNT THE 4 SUPPORTS WITH 2 SCREWS IN EACH SUPPORT (1A). INSERT THE BOTTOM SCREW FIRST, TIGHTEN WELL. THEN COMPLETE WITH SIDE SCREW. MAKE SURE THAT THE SCREWS ARE FIRMLY SECURED. UNFOLD THE PIPE FRAMEWORK. MAKE SURE THAT THE STEEL ROPES ARE FULLY STRETCHED (1B).

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
ATTACH THE SUSPENSION DEVICES (2A), ONE ON EACH SIDE, WITH 3 SCREWS AND SPRING WASHERS ON EACH SIDE.

(NOTE: THE STOP PIN (2B) SHALL BE TURNED TOWARDS THE HEAD END OF THE TABLE).

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
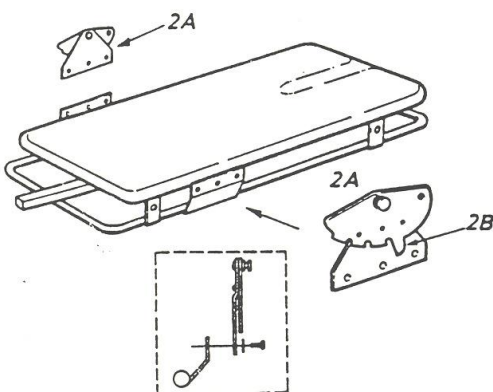
PUT THE BACKREST IN ITS POSITION IN THE PIPE FRAMEWORK (3).

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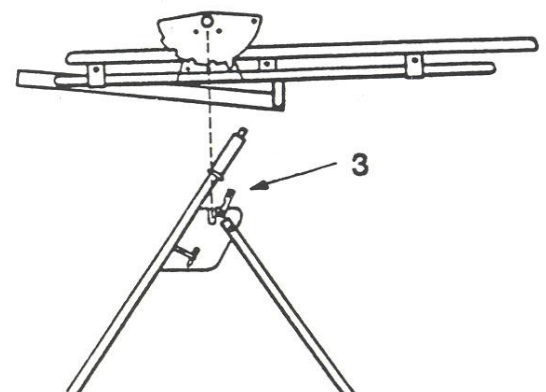
FASTEN THE LOCK PLATES (4A). DO NOT OVER TIGHTEN, IT MIGHT DAMAGE THE THREADS.

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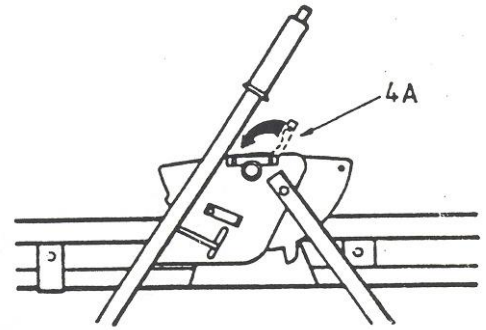MOUNT THE 2 ENCLOSED PADS/FOOT ROLLS ACCORDING TO THIS INSTRUCTION.

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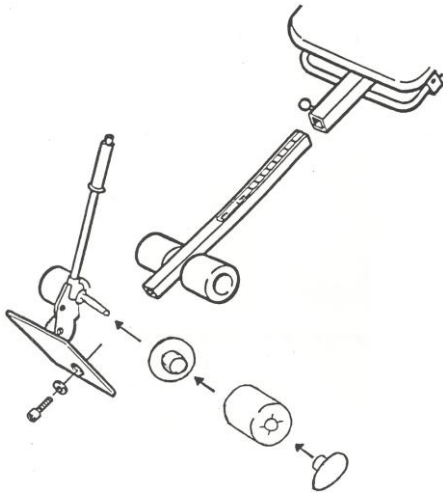
PULL THE BLACK KNOB OUT (5A) AND INSERT THE SQUARE SECTIONED PIPE (5B).

(NOTE: THE LENGTH SCALE MUST POINT UPWARDS).

SECURE THE FOOT PLATE BY SCREWING IT TO THE BOTTOM OF THE SQUARE SECTIONED PIPE.

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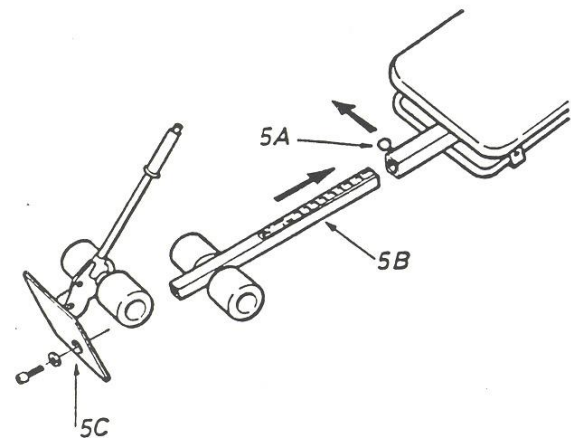IMPORTANT!

ALWAYS CHECK AND MAKE SURE THAT ALL SCREWS AND NUTS ARE SECURELY FASTENED AND THAT THE "FORK" OF THE LOCKING DEVICE IS WORKING PROPERLY/PERFECTLY.

MASTERCARE MUST BE USED ON A HORIZONTAL SURFACE ONLY! READ THE INSTRUCTIONS CAREFULLY BEFORE YOU START USING YOUR MASTERCARE - BACK-A-TRACTION.

P.S.

IF THERE IS SOMETHING THAT YOU DON'T UNDERSTAND, PLEASE CONTACT YOUR DEALER FOR FURTHER INFORMATION AND HELP.

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Addition to the Assembly instructions!

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Master-care®
The Swedish Back Care System™
Back-A-Traction®

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(CN-A1, CN-A2, CN-A3, CN-AH6)

Important!

Before you use Back-A-Traction, model CN-A, the enclosed support must be attached under the foot-plate (A), as shown in the drawing.

Remove the two nuts (B) and put the support (C) in position, as shown in the drawing. Replace the nuts and firmly tighten.

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