



Assembly Instructions: Support Leg, SL1

Accessory to Mastercare Back-A-Traction, Model A

Tools needed: wrench



Place the backrest in horizontal position. Undo the screws (A).
Place the Support Leg in the middle of the tubular frame, underneath
the cut-out for the face in the backrest (B), see above pictures.

B

Fasten the screws in place, the Support Leg will hang straight down.
The Support Leg can hang loose ready for use (C)...or can be tightened,
with the handle, in order to hang underneath the backrest (D) when not in use
(see pictures below).



Always use the handle (1) – never pull the Support Leg itself !



C



D



1